

Turbo Elite Catheter

NES Soiled Device Return Instructions

Step 1

Wipe everything but the barcode label of the device with a cloth and 70% alcohol.

DO NOT immerse the device in water or alcohol.



Step 2

Flush the lumen with a syringe full of 70% alcohol.

Then flush the lumen with a syringe full of air to remove as much liquid as possible.




Step 3

Carefully place the device back into the original protective packaging (coil, tray, etc.) to avoid kinking or damage.



Step 4


Place the packed device into the plastic bag.

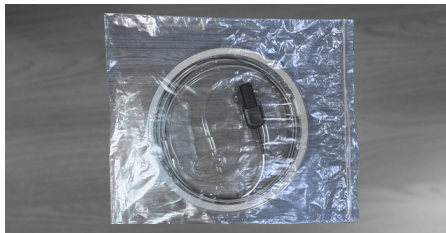

Step 5

Place the device into a sturdy shipping box, filling any void with paper or soft material.

Print and fill out a Device Return Form and place inside the shipping box on top of the devices.

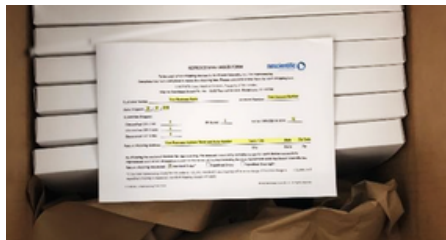

Step 6

Affix the prepaid FedEx shipping label to any corner on the shipping box top and call FedEx at 888-777-6040 to pick up your shipment.



Finding and Printing a Prepaid FedEx Label

To obtain a prepaid FedEx shipping label for returning your device, please visit www.nescientific.com/device-return and click on "Get a shipping label from FedEx." When prompted to enter a Reference Number, enter your customer number, or "NA" if you do not know it. After completing the required information, follow the on-screen instructions to generate and print your prepaid shipping label. If you are returning another type of catheter, please follow that device's specific handling instructions.

Please Note

No biohazard markings are to be placed on the inner or outer packaging. Reference Expectations 49 CFR 173.134(b)(1)(5).